

#### 4.Injectable technique for artificial gum

**Materials used:** A-Silicone for Laboratory, A-Silicone for Gingival Mask



Reminders	
For storage	Sealed and stored in cool place, and storage temperature is 5-25℃.
For shelf life	2 years
For use	<div>① After taking base or catalyst, put the lids on tightly, and the lids should not be interchangeable.</div> <div>② This product is duplication material for dental laboratory use only, which should be kept away from children.</div> <div>③ Waste silicone after taken model should be treated centralized.</div> <div>④ To the allergic individuals, polysiloxane may cause inflammation or other allergic reactions.</div> <div>⑤ The product is for single use.</div> <div>⑥ Do not use after expiration date.</div>

#### Find more about related HUGE products



##### - GumEasy™ A-Silicone for Gingival Mask -

Addition cure silicone for gingival morphology reproduction



##### - Synthetic Polymer Teeth -

Highly esthetic artificial teeth for denture fabrication



##### - Denture Base Polymers -

Esthetic and pliable denture base material for denture base fabrication



#### Shandong Huge Dental Material Corporation

Add / No.68 Shanhai Road, Donggang District, Rizhao City, Shandong Province, 276800, P.R. China.  
Tel / +86 (633) 2277268   marketing@hugedental.com   www.hugedental.com

Facebook Huge Dental

Instagram Huge Dental

Youtube Huge Dental

# HUGE



## A-Silicone for Laboratory

# Alpha<sup>TM</sup>lab

### Duplication Silicone Material

A-Silicone for Laboratory is an addition-curing laboratory kneading silicone recommended for duplicating various models in dental restoration scenarios. The product is characterized by high precision, reliable dimensional stability, suitable final hardness and easy operation.



Alpha<sup>TM</sup>lab

## A-Silicone for Laboratory

### Benefits:



- Easy mixing ratio 1:1
- High fitting and outcoming precision
- Smooth surface after curing
- High detail replication
- Reliable dimensional stability over time
- Delicate hand feeling
- No irritants and nasty smell
- Resistant to high temperature

### Applications:

- Duplicating complete or partial denture models
- Making temporary prosthetic works
- Creating artificial gingiva on the model
- Matrix for esthetic veneer restoration

Technical features					
Mixing ratio	Mixing time*	Total working time*	Setting time*	Hardness	Color
1:1	30s	1 min 30s	8 min	Shore A 80/Shore A 85	Blue/Light Blue

\* The specified times may vary depending on the operating temperature and technique.

Packaging	
Types	Description
Standard tub	 (5kg tub Base+ 5kg tub Catalyst )
Standard can	 (450g can Base + 450g can Catalyst)

## USER'S GUIDE

Easy and Precise Duplication



A-Silicone for Laboratory is conceived to duplicate dental models in various dental restoration work. It is developed to simplify technician's work with its high-performance properties like easy operation, high precision and high dimensional stability, etc.

### 1. Injectable technique for temporary restoration

Material used: A-Silicone for Laboratory



### 2. Indirect aesthetic temporary restoration

Material used: A-Silicone for Laboratory



### 3. Injectable Technique with A-Silicone for Laboratory for Removable Full Denture

Materials used :

A-Silicone for Laboratory, Synthetic Polymer Teeth, Denture Base Polymers

